


# 1-piece Urostomy



Dansac NovaLife TRE™ ostomy products have received official dermatological accreditation from the Skin Health Alliance, which may bring confidence and reassurance to those who use our products.

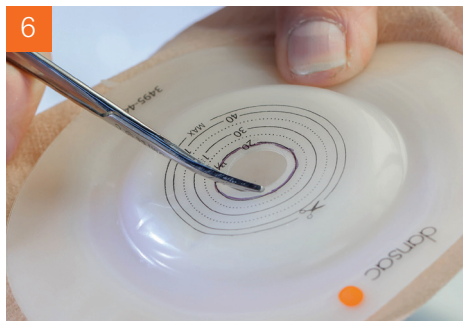

# 1-piece Urostomy



For further information or samples about our pouches,  
visit us at [www.dansac.co.uk](http://www.dansac.co.uk)

**WARNINGS:** Reuse of a single use product may create potential risk to the user such as poor adhesion. Reprocessing, cleaning, disinfection, and sterilisation may compromise the product. This may result in odour or leakage. Ensure proper fit as instructed. If the barrier opening is incorrect or leakage occurs, skin damage may occur (e.g., irritation, broken skin, rash, or redness). This may decrease wear time.

**Notes:** Please note that the individual represented on the back of this material is not the same individual representing the application of the product. To help ensure proper adhesion, apply to clean, dry intact skin. Do not use oils or lotions. These products may interfere with adhesion. Excess mucus from a urostomy stoma may cause the pouch to drain slower than expected. Product fit is affected by many factors such as stoma construction, stoma output, skin condition, and abdominal contours. Contact a stoma care nurse or healthcare practitioner for help with the ostomy pouching system or skin irritation. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see [www.dansac.com/authority](http://www.dansac.com/authority) or local distributor.

The information contained in this brochure is provided as general information only and is not intended to be medical advice. Please see our website for the most up to date information, as guidance can and does sometimes change. Always follow product Instructions For Use and ask your health care professional for more information.

Prior to use, be sure to read the Instructions for Use for information regarding Intended Use, Contraindications, Warnings, Precautions, and Instructions.

Dansac, the Dansac logo, NovaLife TRE and TRE are trademarks of Dansac A/S. All other trademarks and copyright are the property of their respective owners. © 2023 Dansac A/S

